




TACTICAL
— SUPPLY PAKISTAN —

ACCURACY

AT YOUR FINGERTIPS:

MEASURE illegal drug levels in saliva with **EASE & SPEED ON SPOT**

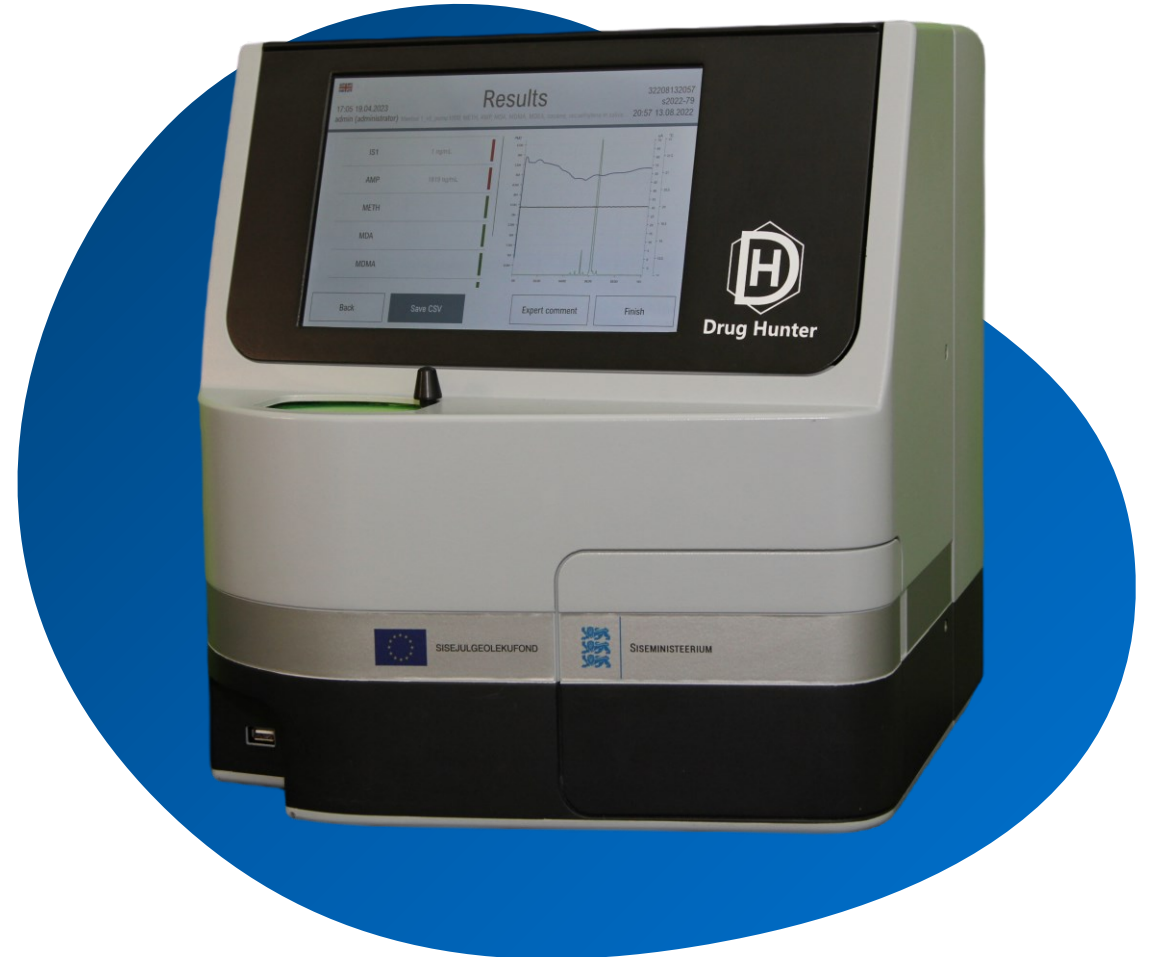

**Drug Hunter & Estonian Police
EUROPOL Excellence Award
Winner 2023**




HOW IT WORKS?


Drug Hunter utilizes

- Liquid-based separation technique known as **CAPILLARY ELECTROPHORESIS** to separate a complex sample matrix into its components
- And registers **FLUORESCENCE** signal from illegal drugs in saliva

POSITIVE SAMPLE

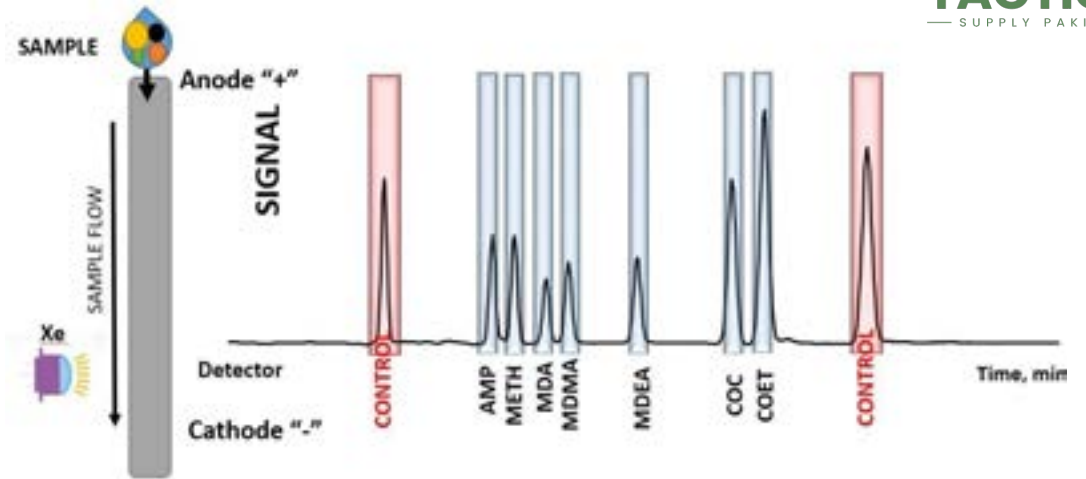


NEGATIVE SAMPLE

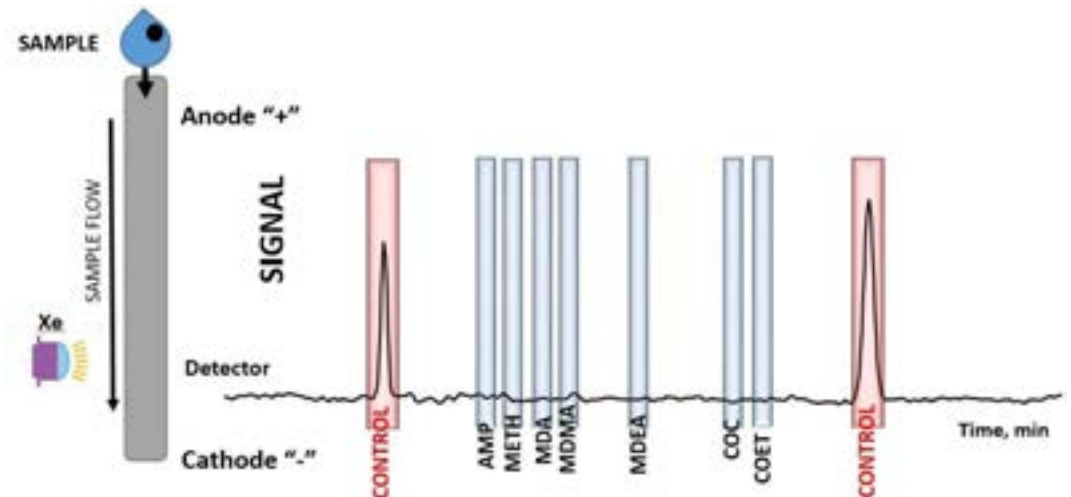

20s



1.5 min

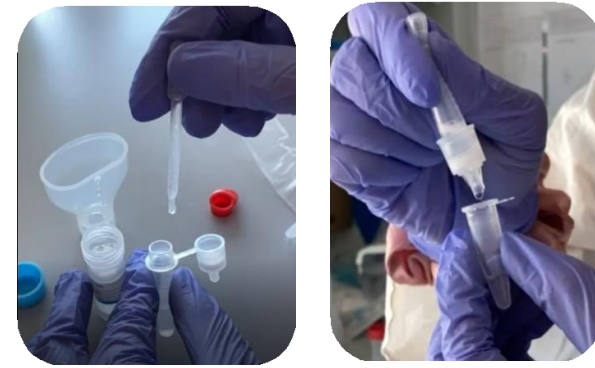

8 min



How to interpret the results?



Cocaine use



METH & MDMA (Ecstasy) use

Method 1 – Negative sample




09:38 07.05.2023
admin (administrator)

Results

M1_3230501v1: METH, AMP, MDA, MDMA, MDEA, cocaine, cocaethylene, morphine in saliva (10 min)

ctrl
Blank saliva
19:14 02.05.2023

IS1	1 ng/mL	
AMP		
METH		
MDA		
MDMA		

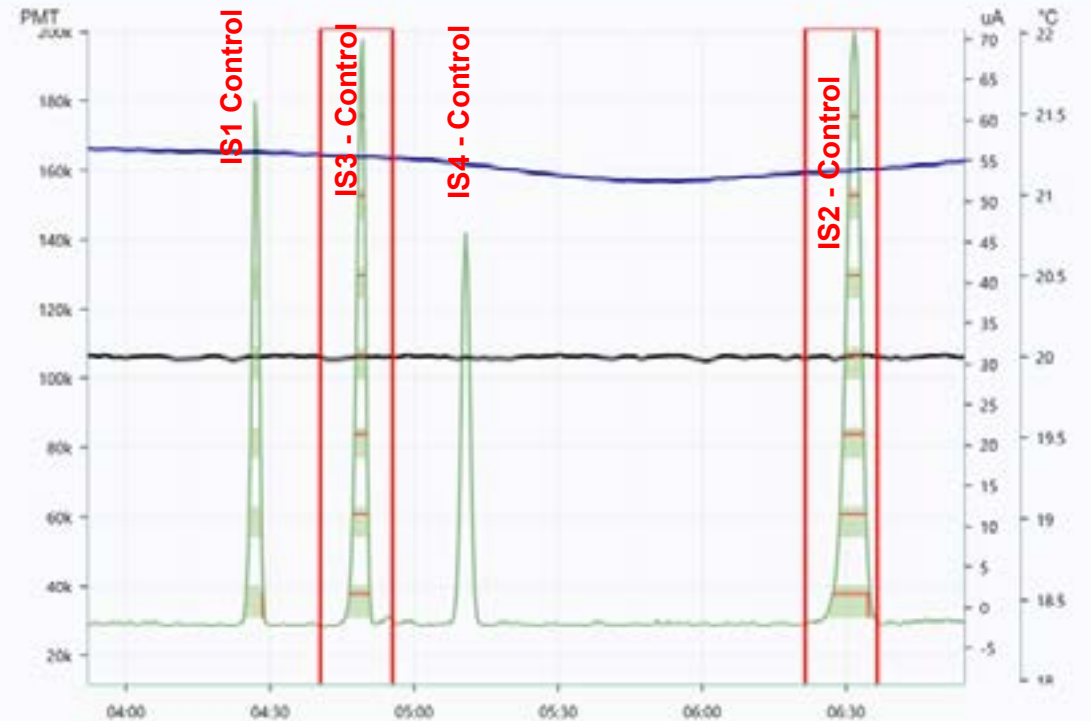

Back

Expert comment

Save CSV

Save

Recal

IS3
<< >>

Finish

Method 1 - Positive sample




09:52 07.05.2023
admin (administrator)

Results

M1_3230501v1: METH, AMP, MDA, MDMA, MDEA, cocaine, cocaethylene, morphine in saliva (10 min)

32302171133
S7_22,5xULOQ_17.02.2023_S
11:33 17.02.2023
Error code: 6

IS1	1 ng/mL
AMP	8944 ng/mL
METH	9116 ng/mL
MDA	498 ng/mL
MDMA	499 ng/mL

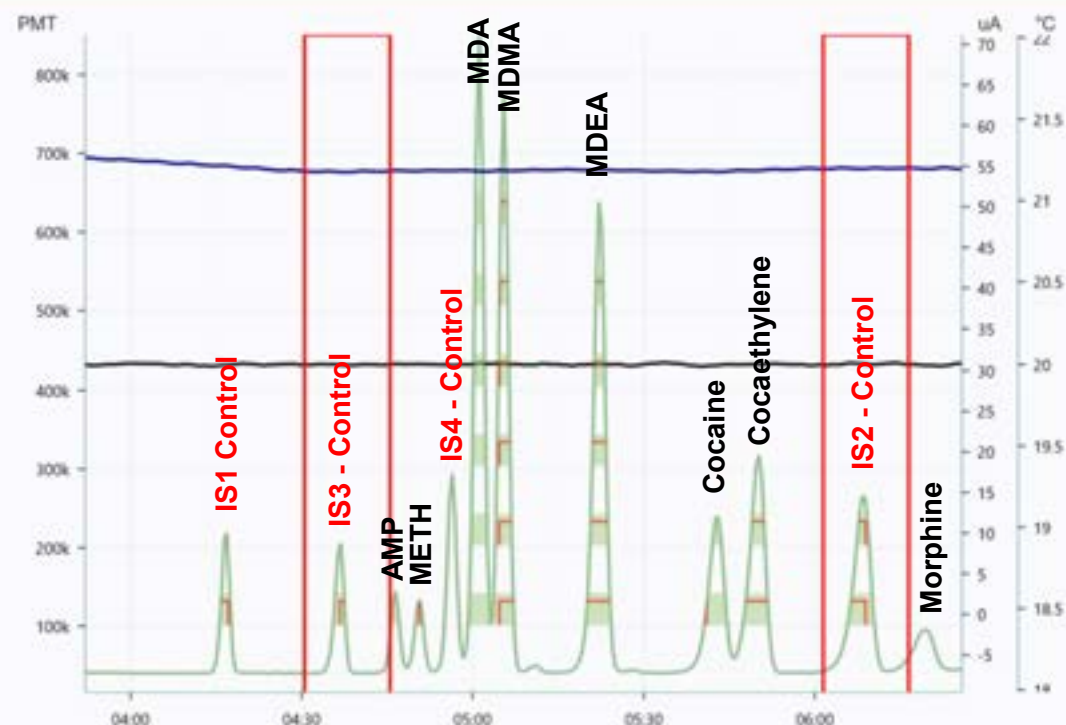

Back

Expert comment

Save CSV

Save

Recal

IS3
<< >>

Finish

Method 2 – Positive sample




09:56 07.05.2023
admin (administrator)

Results

M2_2230502v1: THC

ctrl
-
23:45 02.05.2023
Error code: 6

IS1	1 ng/mL	
THC	2146 ng/mL	

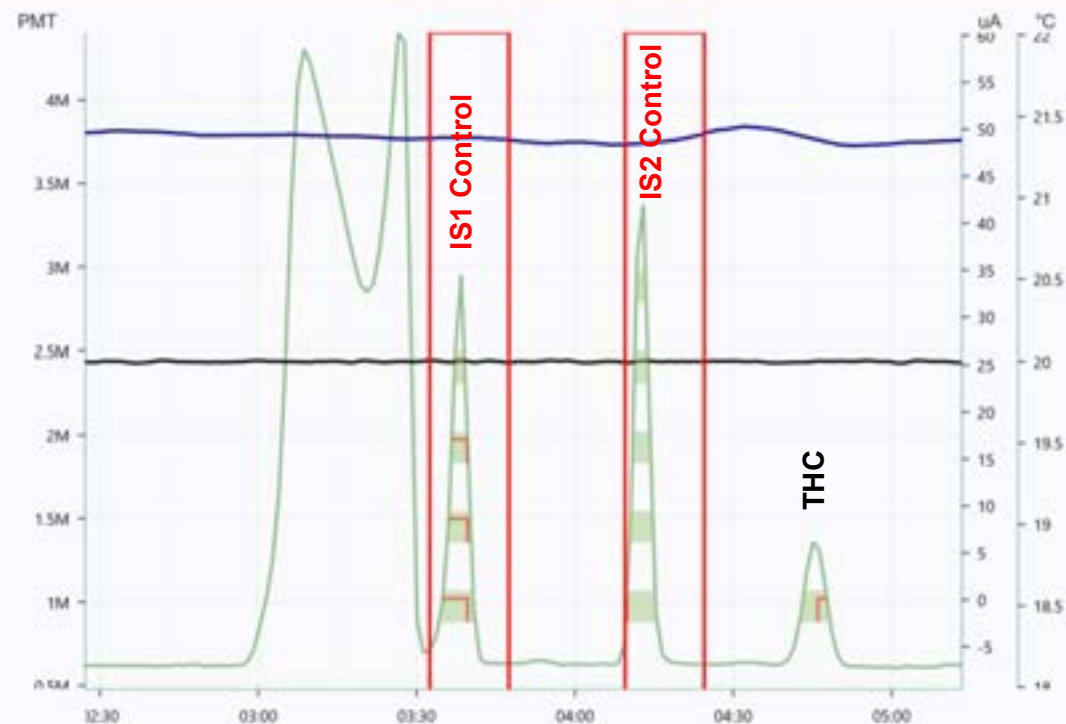

Back

Expert comment

Save CSV

Save

Recal

IS1
<< >>

Finish